



INSTRUCTIONS FOR DISPOSABLE INFUSION SETS

INDICATIONS FOR USE

The product is gravity feed only and used to infusion.

Main structure and properties

- Tubing and drip chamber are transparent, can easy to observe;
- Tubing is flexible enough to resist winding and twisting;
- The design of closure-piercing device to make it convenient for handle and usage;
- Flow regulator can make the flow rate change from minimum to maximum.

USAGE

- Firstly, check valid period and completeness of package before using;
- Tear the package at the cut part and take out the infusion set;
- Close the regulator, pull out protective cap of closure-piercing device and push the spike into the disinfection closure of fluid container;
- Put fluid container upside down, press chamber and half of it was filled with injection solution;
- then open the regulator to expel air in tube.
- Penetrate vein according aseptic manipulation after expelling air then fix it.
- Observing and then change flow rate to normal.
- Infusion operation must have the professionals nursing to implement and guardian.
- 20 drops=(1±0.1)ml[(1±0.1)g]

PRECAUTIONS

- It may causes hypothermia-esp, weak patients;
- It may causes abnormal thythem of ventrickes of heart because of fast flow rate;
- There should maintain the guardianship during the infusion in case of any accident;
- It should be sterile and free from pyrogen before package opened;
- It has a valid period of five years, do not use if it is beyond valid date.
- Discard after singe use.

WARNINGS

- Do not use if the package is opened or damaged, or protective cap is aparted, any part is missed, and foreign matter is found;
- It should be used by special trained person, and the skin should be correctly sterilized.
- The products can not be reused. The repetition use may cause cross infection, even serious illnesses such as hepatitis, HIV and other infectious disease.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

DO NOT REUSE	DO NOT USE IF PACKAGE IS DAMAGED	STERILE STERILE	STERILE EO STERILIZED USING ETHYLENE OXIDE
MD MEDICAL DEVICE	UDI UNIQUE DEVICE IDENTIFIER	CAUTION	CE 2862
Single sterile barrier system	Single sterile barrier system with protective packaging outside	REV.: A/0 EFFECTIVE: 2022-01-24	