




INSTRUCTIONS FOR DISPOSABLE SAFETY NEEDLES

INDICATIONS FOR USE

Disposable safety needles can be used in hypodermic injection. It suits users of different age and is the basic methods of injection.

PRODUCT PERFORMANCE STRUCTURE AND COMPOSITION

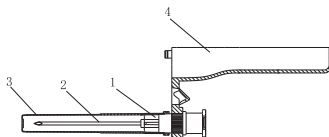

1-Needle Stand ; 2-Needle ; 3-Protective Cap ; 4-Needle Sharp Injury Protection Mechanism .

SPECIFICATION

16G,18G,20G,21G,22G,23G,24G,25G,26G,27G.

USAGE

- It is used together with other medical instrument;
- Check valid period and completeness of package before using;
- Open the package from the peel site and take out the safety needles;
- Connect it with syringes tightly;
- Pull out protective cap of needle. Then do as aseptic manipulation;
- After injection, turn safety shield forward until you feel the positive stop and hear an audible click, so as to make needle fully covered;
- Put into the specified medical waste box.

CONTRAINDICATIONS AND PRECAUTIONS

- Expel all air in barrel from the needle upwardly, in case air be injected into human body;
- It should be sterile and free from pyrogen before package opened;
- After use, this product should be discarded in the specified medical waste collection box, shall not be reused, and should be treated according to the relevant provisions of medical waste disposal;
- It has a valid period of five years, check valid period and completeness of package before using, do not use if it is beyond valid date.

WARNINGS AND INSTRUCTIONS

- Do not use if the package is opened or damaged, or protective cap is aparted, and part is missed, and foreign matter if found;
- It should be used by special trained person, and the skin should be correctly sterilized;
- The products can not be resued. The repetition use may cause cross infection, even serious illnesses such as hepatitis, HIV and other infectious disease.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, temperature ranges from -10 to 40°C.

SYMBOLS

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide
Date of manufacture	Manufacturer	Consult Instructions For Use	Do not re-use	Do not use if package is damaged
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation
Unique sterile barrier system	Simple aseptic barrier system with external protective packaging		REV.: A/0 EFFECTIVE DATE: 2023-11-14	

Anhui Tiankang Medical Technology Co., Ltd.
No. 228, Weiyi Road, Economic Development
Zone, Tianchang City, Anhui Province, P.R. China.

Shanghai International Holding Corp. GmbH(Europe)
Eiffestrasse 80 20537 Hamburg, Germany
Tel:0049-40-2513175

