

INSTRUCTIONS FOR Safety Syringes with Retractable Needles

INDICATIONS FOR USE

The Safety Syringes with Retractable Needles is used for hypodermic injection dose. It is suitable in different age.

Main structure and properties

This product is made up of barrel, plunger, piston, needle and reuse prevention feature.

Re-use prevention feature will start all by itself the moment when the plunger. has been push 100% to the bottom of barrel. And then start its safety function by hand, namely pull back the Plunger with needle, So as to prevent people from being hurt by the needle.

SPECIFICATION

-TKSSR01: 1ml; TKSSR02: 2ml,2.5ml,3ml,5ml,10ml,20ml

-Needle size: 18G to 30G

USAGE

-First, Check valid period and completeness of package before using;

-Open the package from the peel site and get out the syringe;


-Push and pull plunger till it is well lubricated;

-It should be used with needle;

-Remove protective cap, absorb injection solution and use it according aseptic manipulation.

-Start its safety function by hand, namely, pull back the plunger back till it is locked, and then break the plunger.(see picture 1, 2, 3);

-Put it into safety box after single use of it.

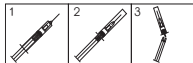

PRECAUTIONS

-Expel all the air in barrel from the needle upwadsly, in case air be injected into human body.

-Do not push plunger to the bottom of barrel, because it will make the product destroyed.

-It should be sterile and free from pyrogen before package opened.

-It has a valid period of five years, Do not use if it is beyond valid date.

-Discard after single use

WARNINGS

-Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found.

-It should be used by special trained person, and the skin should be correctly sterilized.

-The products can not be reused, the repetition use may cause cross infection, even serious illnesses, such as hepatitis, HIV and other infectious disease.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

SYMBOLS

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide	
Date of manufacture	Manufacturer	Re-use prevention	Do not re-use	Do not use if package is damaged	
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit	
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation	
Consult Instructions For Use	Single sterile barrier system	Single sterile barrier system with protective packaging outside		REV.: A/0 DATE: 2023-11-14	