

受控



Instructions for A.V. Fistula Needles

【 Scope of Application 】

- This product is suitable for establishing vascular access in hemodialysis treatment.

【 Main structure and properties 】

- The product is made up of protective cap, needlestand, tubing, on-off clamp, luer lock connector And luer lock endcap;
- This product has a good property of rigidity, toughness and corrosion resistance;
- Tubing is transparent that can easy to observe;
- Tubing is flexible enough to resist winding and twisting;
- On-off clamp can make the blood rate change from minimum to maximum;
- The design of needle stand to make it convenient for handle.



【 Specification 】

- 15G~17G

【 Usage 】

- Firstly, check valid period and completeness of the products before using;
- Open the package from the peel site and take out A.V. Fistula Needles;
- Connecting connector with pipe of machine;
- Pull out needle protective cap and expel air in tubing;
- push the needle spike into the body vein according aseptic manipulation and fix it.

【 Precautions 】

- It may causes hypothermia-esp. Waek patients;
- It may causes abnormal thrythm of ventrickes of heart because of fast flow rate;
- There should be guardian during infusion in case of any accident;
- It should be sterile and free from pyrogen before package opened;
- It has a valid period of five years, Do not use if it is beyond valid date;
- Discard after single use.

【 Warnigs and Indications 】

- Do not use if the package is opened or damaged, or protective cap is aparted, and part is missed, and foreign matter is found;
- It should be used by special trained person, and the skin should be correctly sterilized.
- The products can not be resued.The repetition use may cause cross infection, even serious illnesses such as hepatitis,HIV and other infectious disease.

【 Storage 】

- Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

【 Symbols 】

| | | | | | |
|-------------------------------|--------------|---|------------------------------------|----------------------------------|--|
| Unique device identifier | Batch code | Catalogue number | Contains or presence of phthalates | Sterilized using ethylene oxide | |
| Date of manufacture | Manufacturer | Do not re-use | Caution | Do not use if package is damaged | |
| Medical device | Use-by date | Do not sterilize | Fragile, handle with care | Temperature limit | |
| Keep away from sunlight | Keep dry | Non-pyrogenic | This way up | Humidity limitation | |
| Single sterile barrier system | | Single sterile barrier system with protective packaging outside | | REV.: A/0 DATE: 2023-11-14 | |

Anhui Tiankang Medical Technology Co., Ltd.
No.228, Weiji Road, Economic Development Zone,
Tianchang City, Anhui Province, P. R. China.



Shanghai International Holding Corp. GmbH(Europe)
Eiffestraße, 80 20537 Hamburg, Germany
Tel:0049-40-2513175