



INSTRUCTIONS FOR Disposable Blood Lancets

【Scope of Application】

- The product is intended for making punctures to obtain small blood specimens and to prick the skin testing for allergies.

【main structure and properties】

- The product is made up of protective cap , needle and needle stand;
- This product has a good property of rigidity, toughness and corrosion resistance;
- Protective cap and needle stand match up well, and it is never loosen itself.

【Scope of Application】

- 0.3mm~0.8mm

【usage】

- It is used together with blood taking instruments;
- The way is piercing skin of fingers or foot bottom ;
- Check valid period and completeness of products before using;
- Open the package from the peel site and take out it;
- Pull out protective cap, Then do as aseptic manipulation.

【Precautions】

- It should be sterile and free from pyrogenic before package opened;
- It has a valid period of five years.
- Do not use if it is beyond valid date;
- Discard after single use.;

【warnings and indications】

- Do not use if the package is opened or damaged, or protective cap is aparted, any part is missed, or foreign matter is found;
- It should be used by professional, and the skin should be correctly sterilized.
- The products can not be reused.The repetition use may cause cross infection, even serious illnesses such as hepatitis,HIV and other infectious disease.

【storage】

- Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide	
Date of manufacture	Manufacturer	Do not re-use	Fragile, handle with care	Do not use if package is damaged	
Medical device	Use-by date	Do not re-sterilize		Temperature limit	2862
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation	
Consult Instructions For Use	Single sterile barrier system	Single sterile barrier system with protective packaging outside		REV: A/0 DATE: 2024-01-30	